Respireo SOFT Nasal Non Vented Elbow

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ENGLISH

To properly use Respireo SOFT Nasal Non Vented Elbow, please refer to the instruction of Respireo SOFT Nasal mask.

1 – INTENDED USE

Respireo SOFT Nasal Non Vented Elbow, mounted in place of the elbow connector with calibrated vent ports (c) and the swivel fitting (d) is intended to be used for treating patients with controller non-invasive positive pressure ventilation (NPPV). Respireo SOFT Nasal Non Vented Elbow is suitable for adult patient (>30 kg). Respireo SOFT Nasal Non Vented Elbow is intended for single-patient reuse in the home environment and multiple reuse in the hospital environment.

2 – WARNINGS

- The mask must only be used with a ventilator equipped with an active exhalation valve or a respiratory circuit with an exhalation valve.
 - The operating pressure range is 4-40 cmH₂O.
 - If oxygen is used with the ventilator, the supply must be turned off when the appliance is not in operation.
- For further instructions, please refer to section 2 (excluding points 2.1, 2.2, 2.3 and 2.6) in the instruction booklet for Respireo SOFT Nasal mask.

3 – COMPONENTS

- Elbow connector (c').

4 – HOW TO ASSEMBLE THE KIT (Fig. F-G)

Disassemble the elbow fitting (c) and the swivel fitting (d) from Respireo SOFT Nasal mask (Fig. F) then replace mount respireo SOFT Nasal Non Vented Elbow (Fig.G).

5 – USE AND WARRANTY

Once Respireo SOFT Nasal Non Vented Elbow is in place, please refer to Respireo SOFT Nasal user manual for what concerns:

- HOW TO MOUNT AND FIT THE MASK (paragraph 4);
- REMOVING THE MASK (paragraph 5);
- DISASSEMBLING THE MASK (paragraph 6);
- CLEANING THE MASK (paragraph 7);
- REASSEMBLY THE MASK (paragraph 8);
- WARRANTY, COPYRIGHT, TECHNICAL UPDATES (paragraph 12).

6 – TROUBLESHOOTING

Please refer to section 9 in the instruction booklet for the Respireo SOFT Nasal mask. The vents are not present on SOFT Nasal Non Vented Elbow (c').

7 – DISPOSAL INSTRUCTION

Respireo SOFT Nasal Non Vented Elbow (c') is made of polycarbonate.

8 – TECHNICAL DATA

Please refer to section 11 in the instruction booklet for the Respireo SOFT Nasal mask. The PRESSURE-FLOW CURVE, RESISTANCE, SOUND PRESSURE LEVEL AND SOUND POWER LEVEL data shall not be considered as the elbow connector with calibrated vent ports (c) is not used.

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